

Splint process

1. Take impressions and place an order

Impression with alginate



Pour the impressions



Ready to send



- 1) Send **Stone Models** to 450 W 910 S suite 300, Heber City, UT 84032 / Attn: Rosie Kim
- 2) Order.dionavi.com -> Order -> FullArch(Fixed solution)&Prosthetics -> Order Type

Orders

Surgical guide(Incl. edentulous)


Prosthetics

Full arch(Fixed solution) & Prosthetics


Order status

Drill Sequence Search

03 Order Type

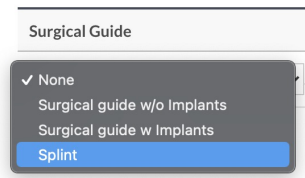

2. After receiving the Splint

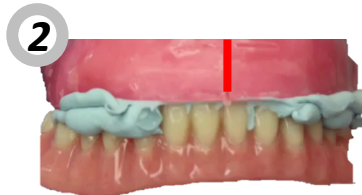

Facial pictures

- > Front
- > Front Smile
- > Front retract
- > Lateral

Intraoral Pictures

- > Front
- > Lateral



[Protocol]

- 1) Splint try in (Wax rim adjustment)
- 2) Mark Middle line and bite registration
- 3) CT scan with Splint (Close Bite)
- 4) Take Facial, Intraoral pictures
- 5) Check teeth, gum shade

Send **Splint, Bite registration** to 450 W 910 S suite 300, Heber City, UT 84032 /Attn: Rosie Kim
Order order.dionavi.com (Order for surgery guide)