

DIOnavi Surgery simulation

Orders

Surgical guide(Incl. edentulous)
Prosthetics
Full arch(Fixed solution) & Prosthetics
Order status
Dashboard(Patterson)
Dashboard(Burkhart)
Drill Sequence Search

12. Shipping (284)

Order Seq	Clinic Name	Clinic Div	Order Date	Order Div	FullArch Div	Patient Name	Impression Type	Order Status	FullArch Designer	View Details	Simulation	Tracking Number
<input type="checkbox"/> 27541		USA	12-08-2022	DIONavi and Immediate restoration	N/A		PLY / Open STL(Export File upload required*)	Shipped	N/A			1Z017A510390806529

Click **Order Status – Shipping - Simulation**

Surgical Precautions

US-ORD-20221208-00016(L0)
12-20-2022
Date of Surgery

Surgical Kit

Surgical Precautions

- 1. Read the instructions for use (IFU) for each surgical instrument. Do not use the instrument for purposes not intended by the manufacturer.
- 2. The instrument is designed for use in the oral cavity. Do not use the instrument for purposes not intended by the manufacturer.
- 3. Do not use the instrument for purposes not intended by the manufacturer.
- 4. Do not use the instrument for purposes not intended by the manufacturer.
- 5. Do not use the instrument for purposes not intended by the manufacturer.
- 6. Do not use the instrument for purposes not intended by the manufacturer.
- 7. Do not use the instrument for purposes not intended by the manufacturer.
- 8. Do not use the instrument for purposes not intended by the manufacturer.
- 9. Do not use the instrument for purposes not intended by the manufacturer.
- 10. Do not use the instrument for purposes not intended by the manufacturer.

Click **Surgery Preview**

It will show what surgical kit you need for this surgery

Surgery Preview

US-ORD-20221208-00016(L0)
12-20-2022
Date of Surgery

Surgical Kit

Instrument	Quantity
U1	1
D1	1
D2	1
D3	1

Click **Locate tooth position**

Locate tooth position

US-ORD-20221208-00016(L0)
12-20-2022
Date of Surgery

Surgical Kit

By clicking on implant information or serial number for respective tooth position, one can choose the right tooth. During process the surgery, dentist can register the tooth to make multiple choices.

Click **tooth number** that you would like to check

Locate tooth position

US-ORD-20221208-00016(L0)

U1
U1004508

1 / 15

10
LNN 18
150 RPM
Tissue Punch

NEXT

11
LNN 18
100 RPM
Bone Flattening Drill

Offset 12mm
D1

18
Normal Case

Click the **play button**